


ARTISEAL™

ArtiSeal Vessel Sealing System

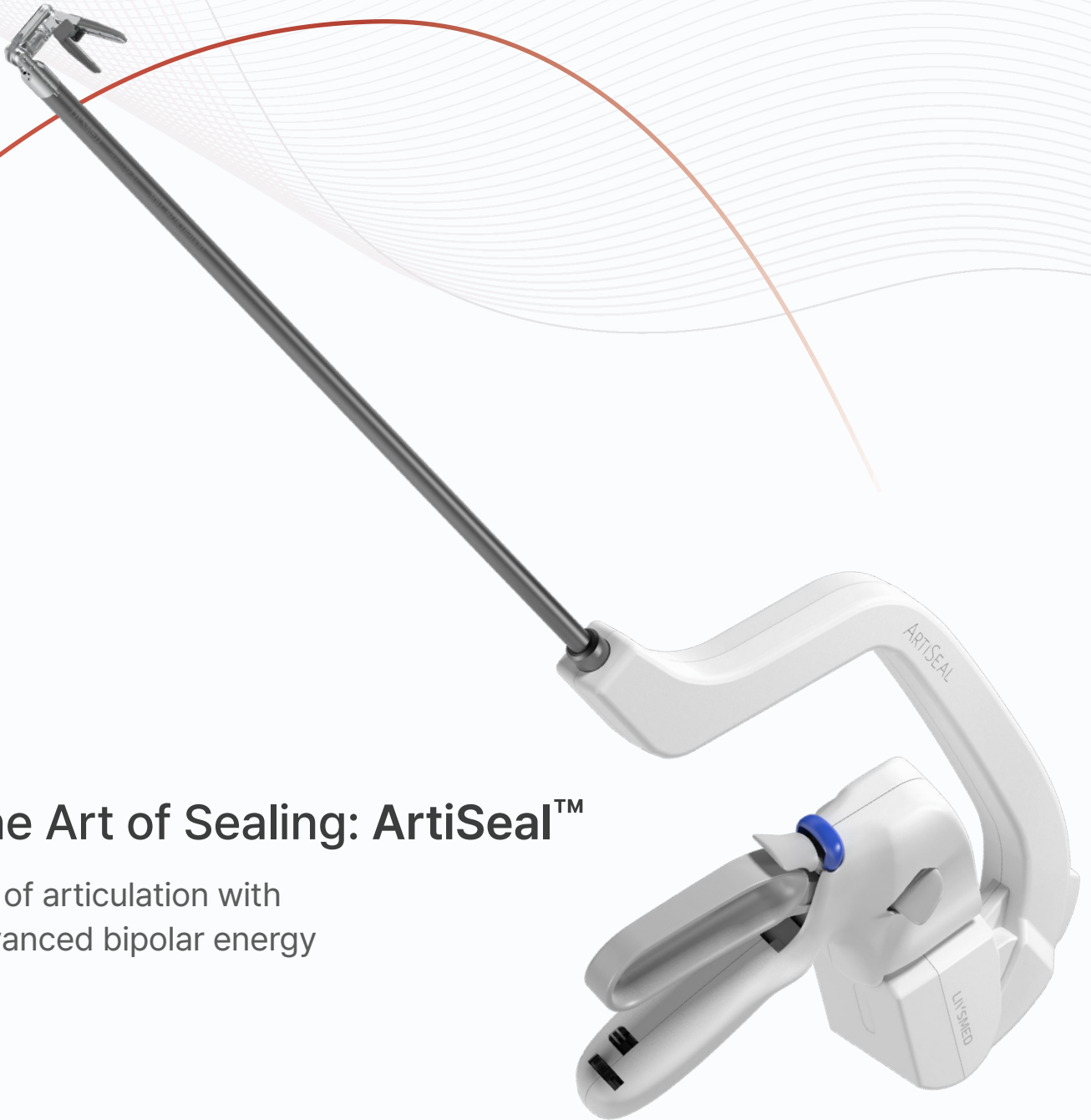

The Art of Sealing: ArtiSeal™

90° of articulation with
advanced bipolar energy

LIVSMED




ArtiSeal™


Full articulation in your hands

ArtiSeal is a new standard in vessel sealing system with 90° of articulation that allows perpendicular access to targeted vessel and tissue bundles.

ArtiSeal Instruments

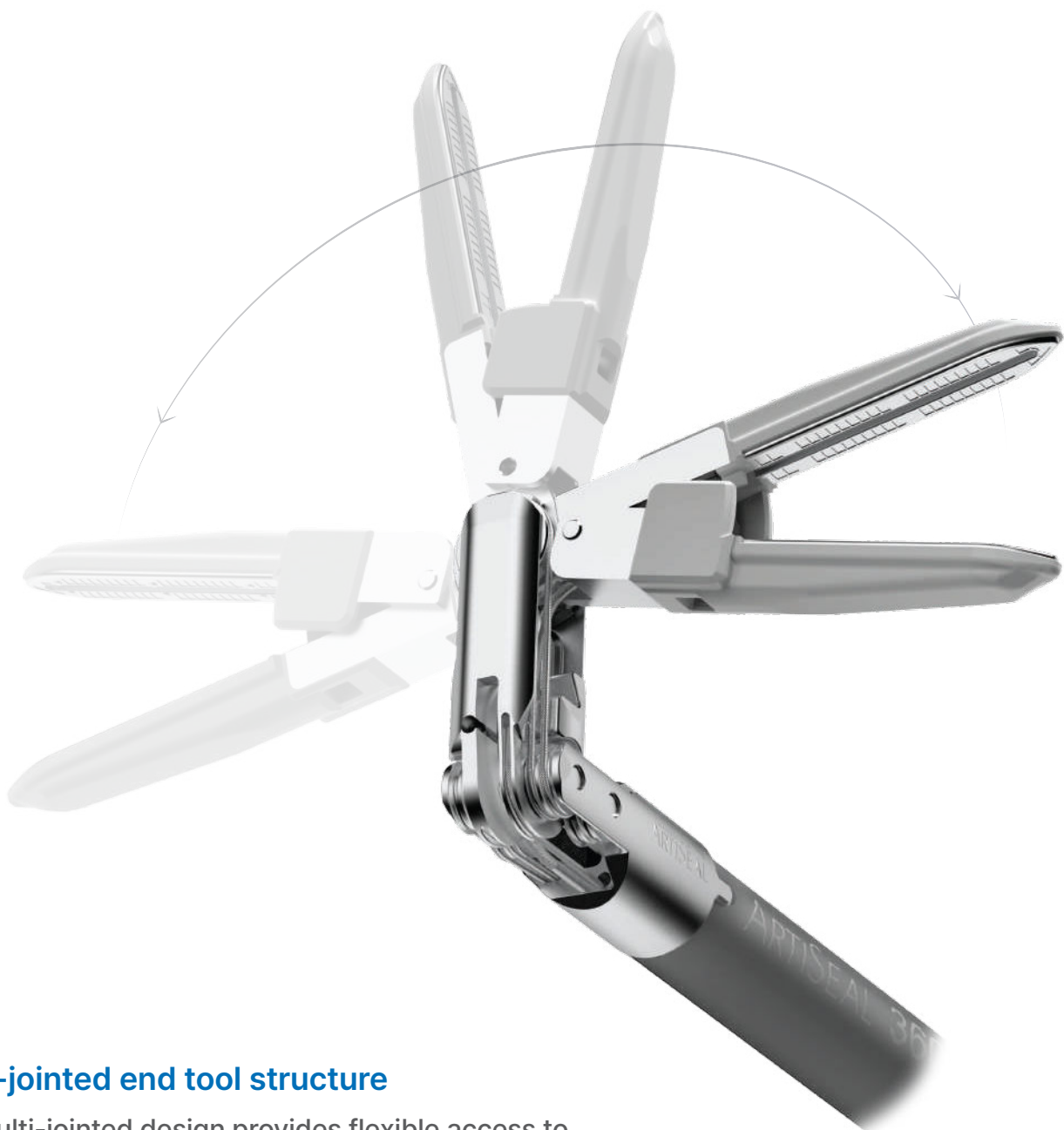


Multi-jointed end tool structure
Our multi-jointed design provides flexible access to the surgical site from multiple angles.
(Jaw length : 21mm)

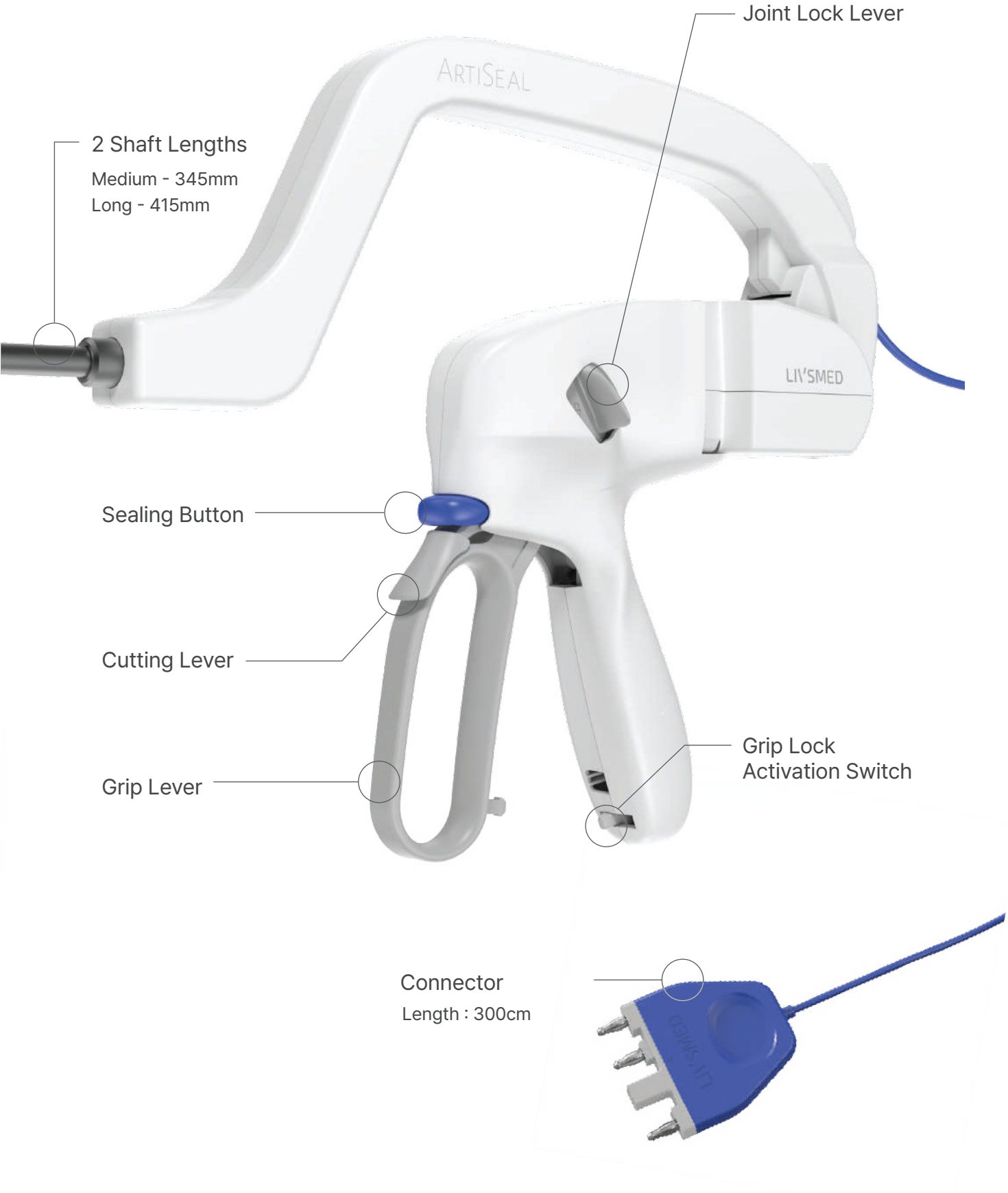


Synchronized Interface

The end tool moves in perfect synchronization with the surgeon's natural wrist and hand movements as conveyed through the device's ergonomic grip.

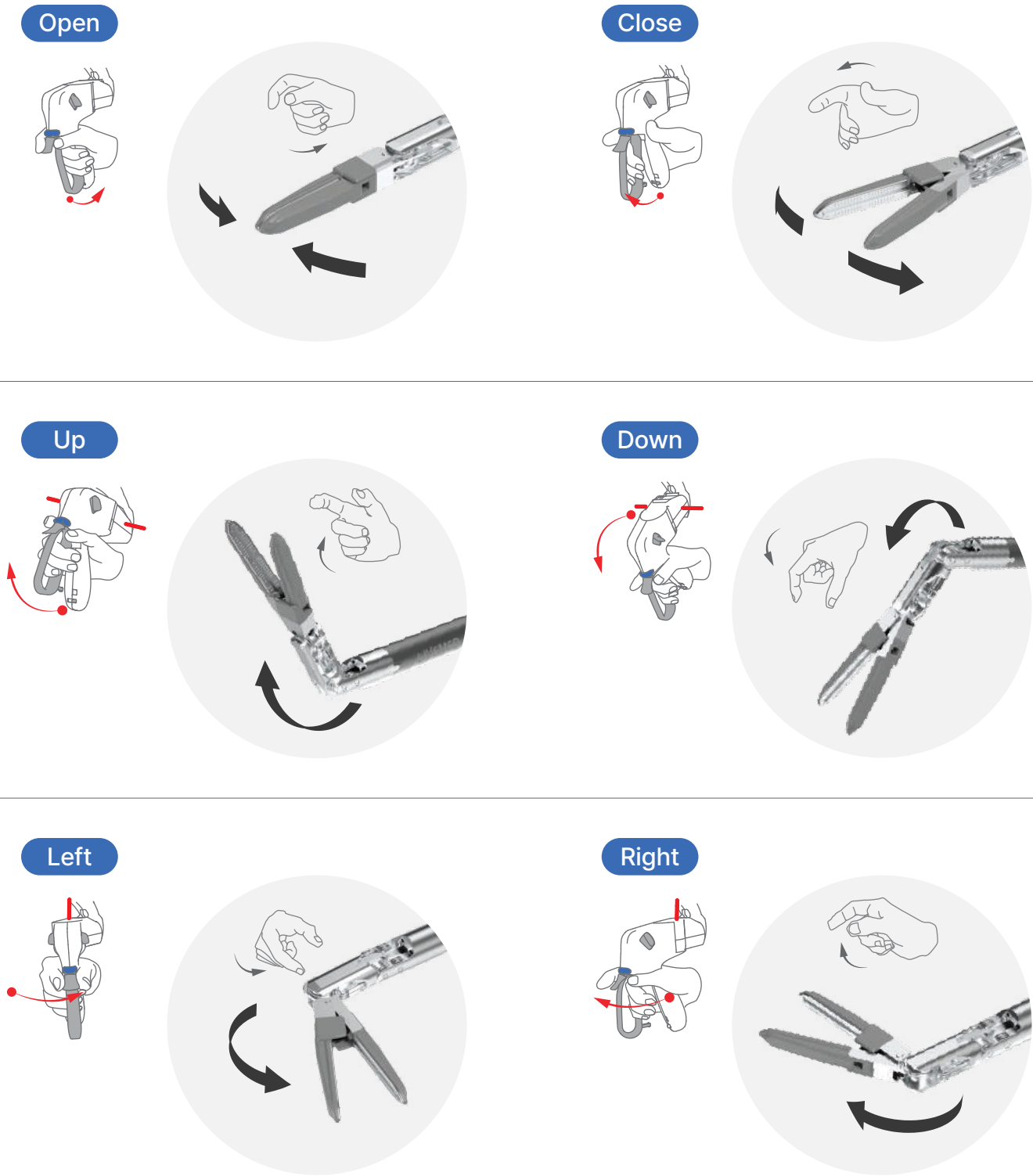


Using ArtiSeal

Sealing Button & Cutting Lever

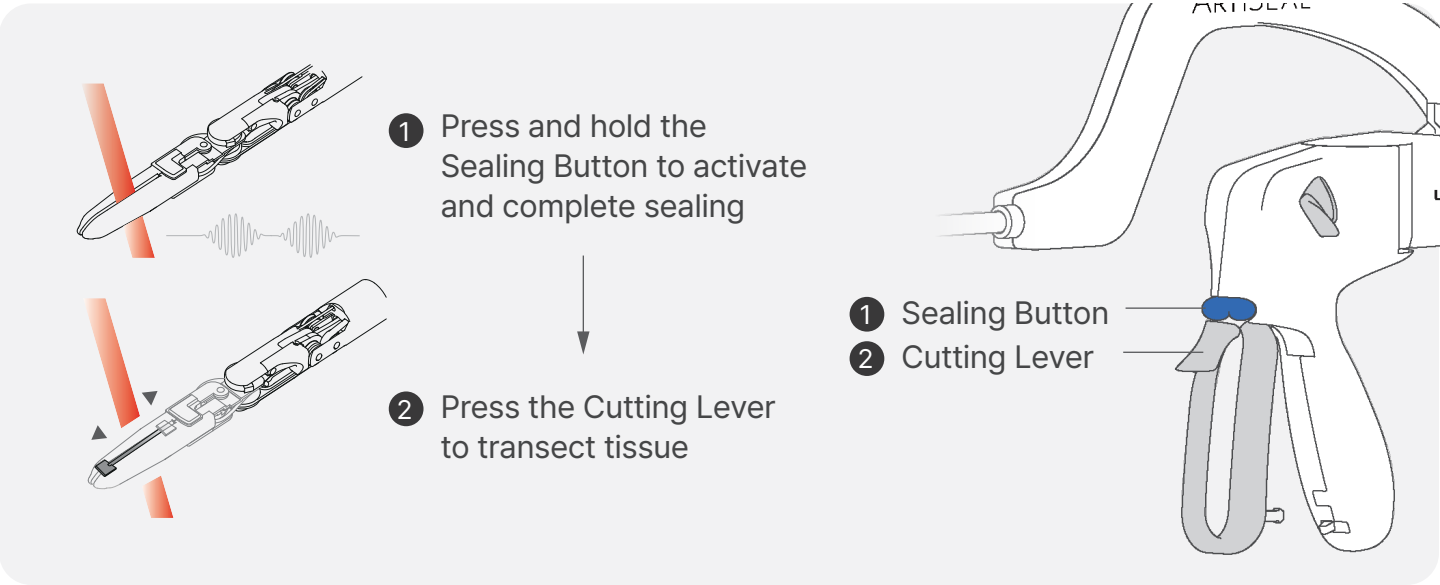


Joint Lock Lever

Lock lever is used to fix the jaws in locked-position

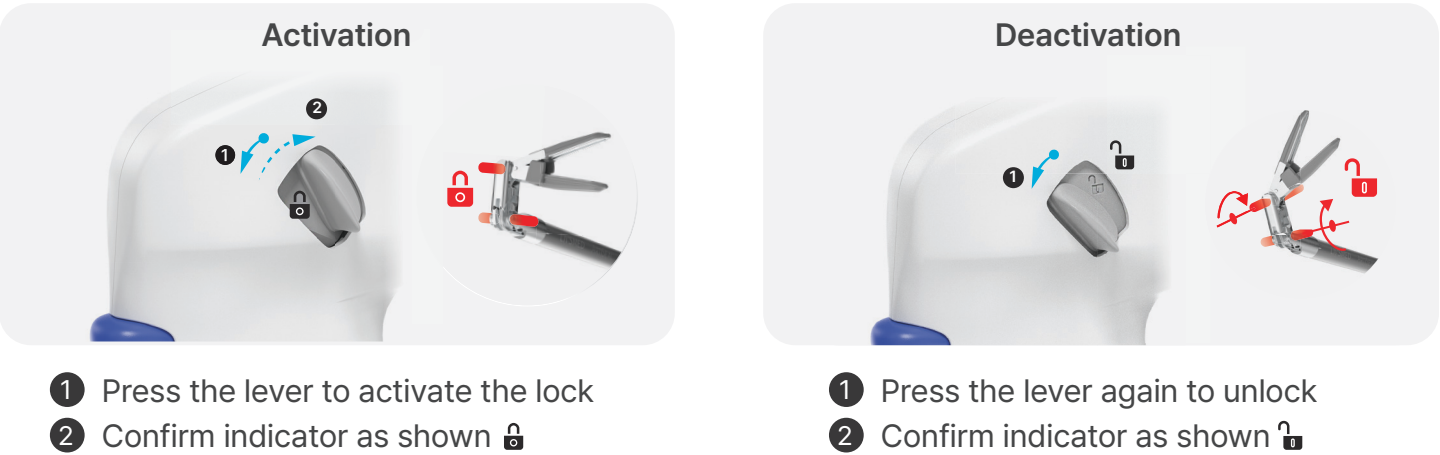

Grip Lever

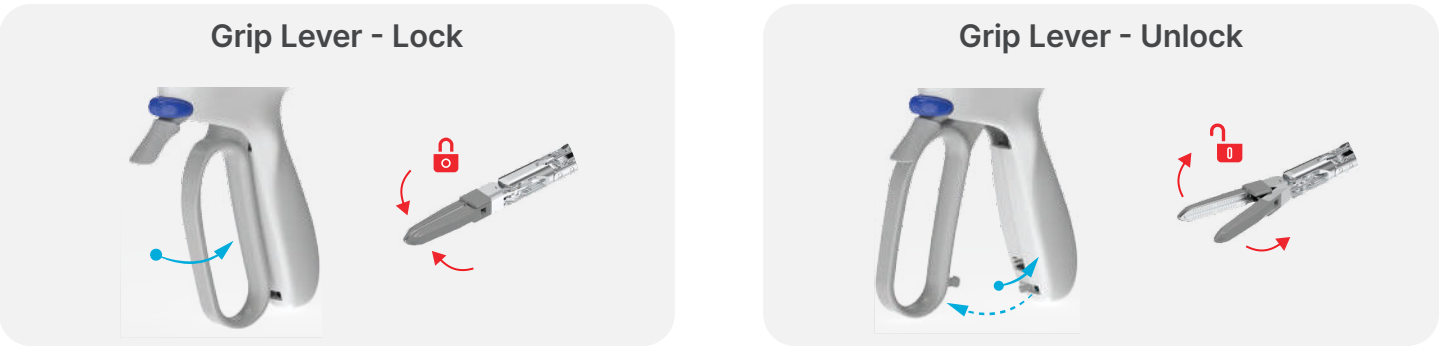

The ArtiSeal Generator


The ArtiSeal Instrument’s dedicated, high-frequency RF generator is specifically designed for vessel sealing and is software upgradeable with built-in safety features.




Quick Guide

ArtiSeal Generator Settings

1. Display Brightness

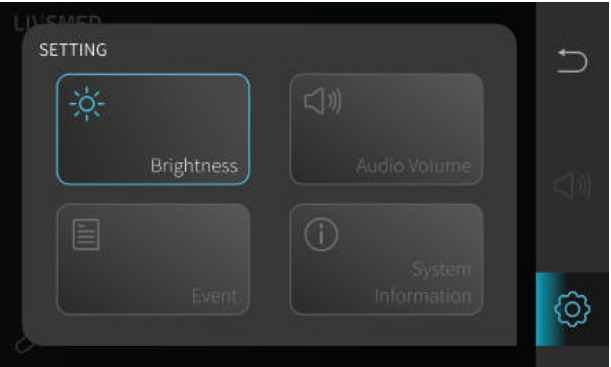


- 1 Select the Brightness option button on the Settings page

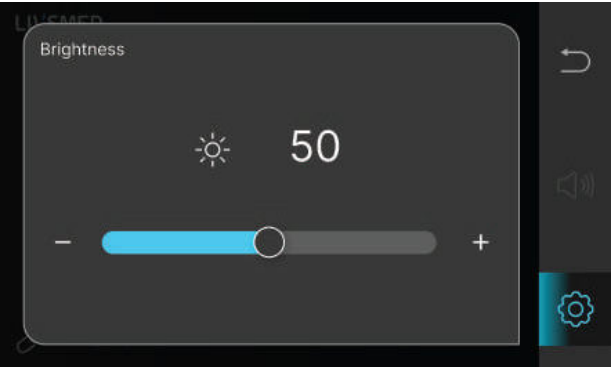


- 2 While pressing the Brightness bar, drag it to the desired brightness level
- 3 Press the Return button

2. Audio Volume

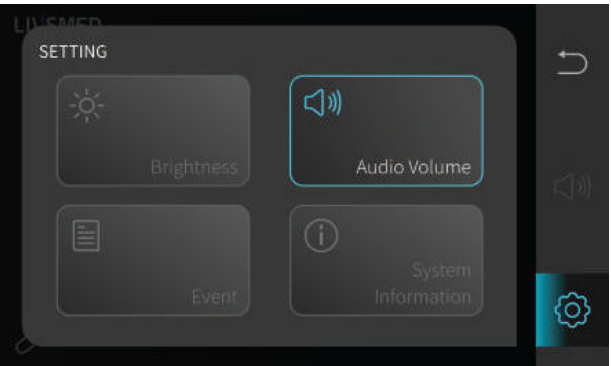


- 1 Select the Audio Volume option button on the Settings page

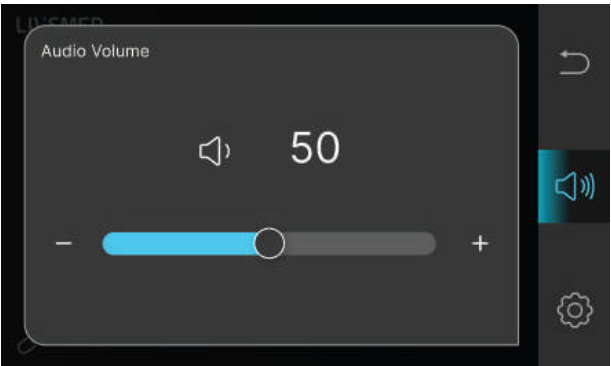


- 2 While pressing the Audio Volume bar, drag it to the desired volume level
- 3 Press the Return button

ArtiSeal Generator Settings

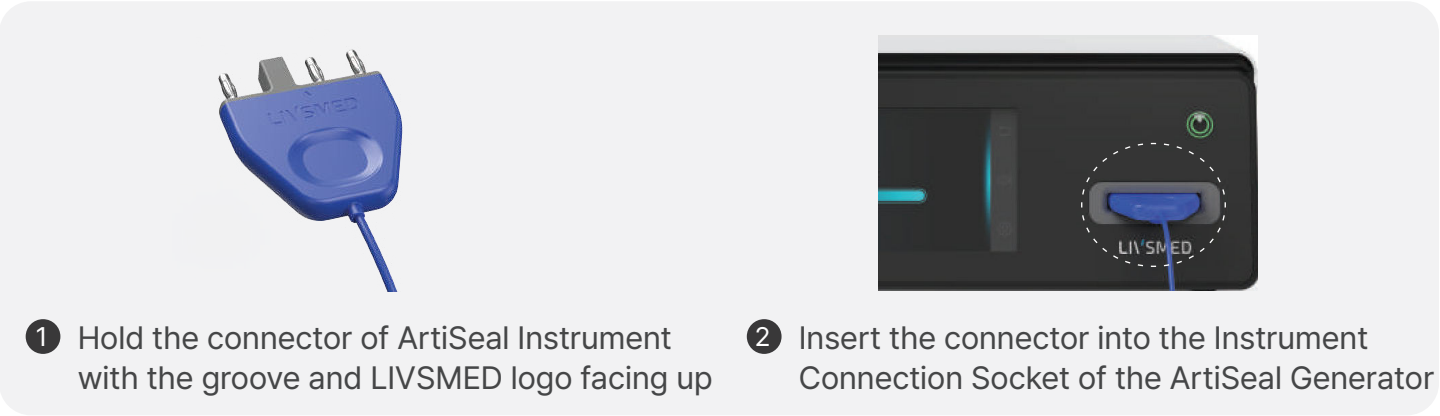


Advanced Articulation

The ArtiSeal Instruments allow perpendicular access to vessels, enabling optimal angles for sealing and cutting. With ArtiSeal instruments, surgeons can get a preferred, precise perpendicular seal.

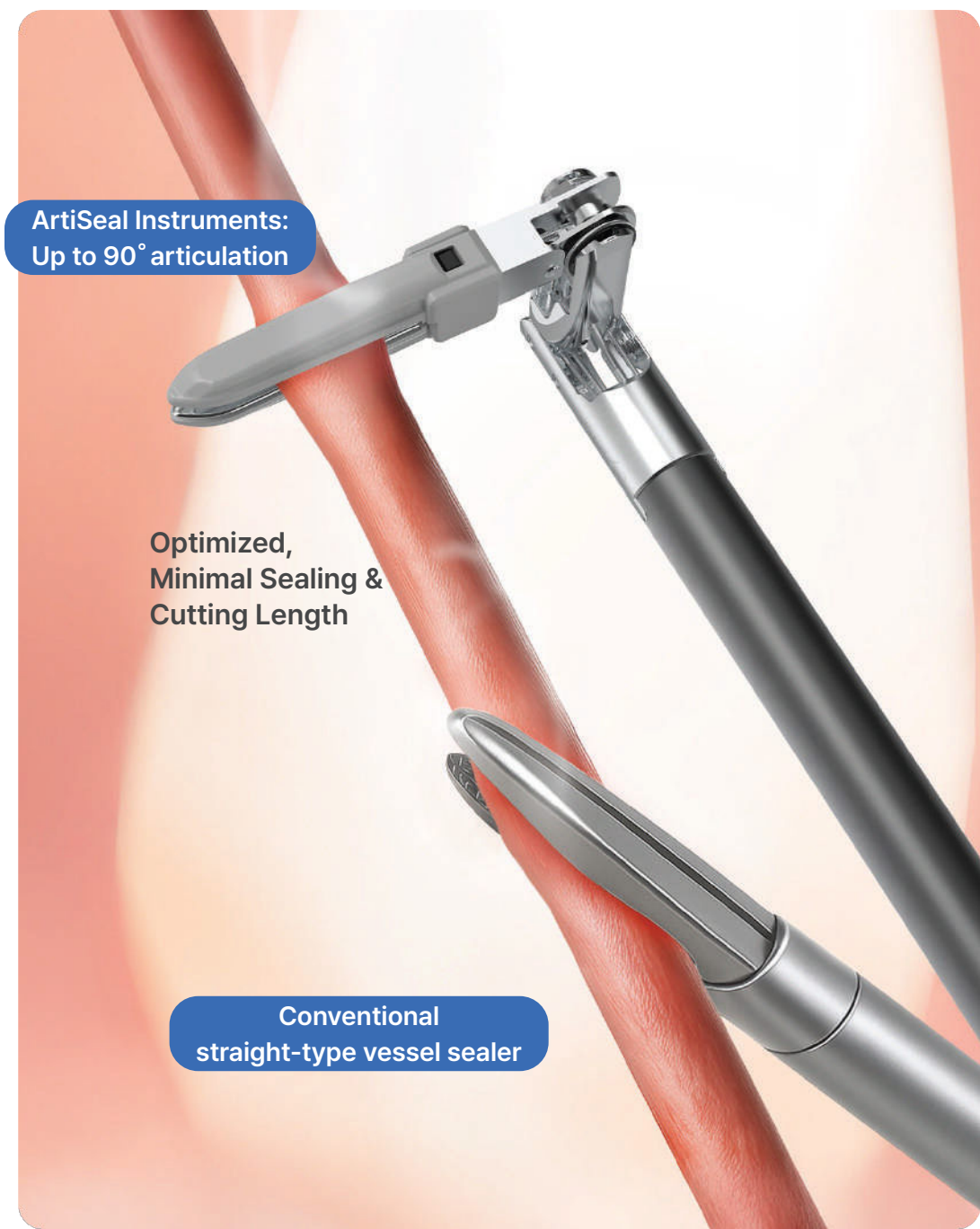

Specifications

ArtiSeal Instruments

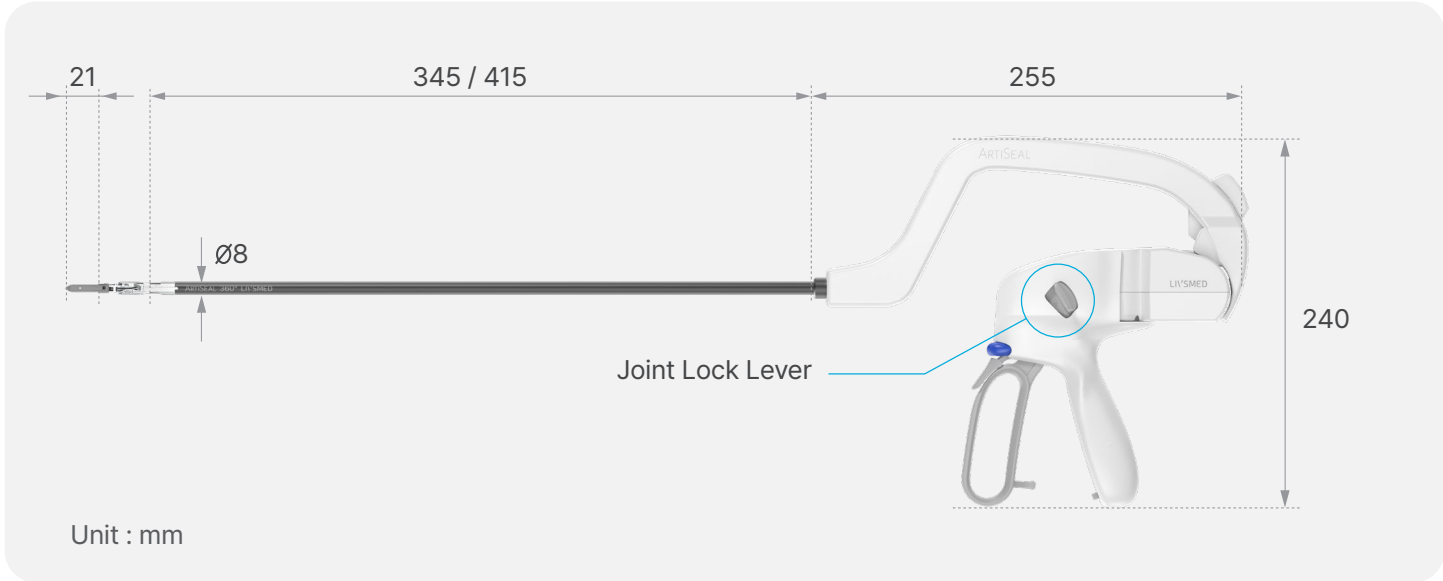


ITEM CODE	DESCRIPTION	JAW LENGTH	SHAFT DIAMETER	ELECTRODE	UNITS/CASE
AS01M	ArtiSeal Instruments-345mm-Lock	21mm	8mm	Bipolar	5EA/CASE
AS01L	ArtiSeal Instruments-415mm-Lock				

ArtiSeal Generator



ITEM CODE	DESCRIPTION	WIDTH	HEIGHT	LENGTH	ELECTRODE	UNIT/CASE
ASG-1000	ArtiSeal Generator	350mm	164mm	455mm	Bipolar	1EA/CASE

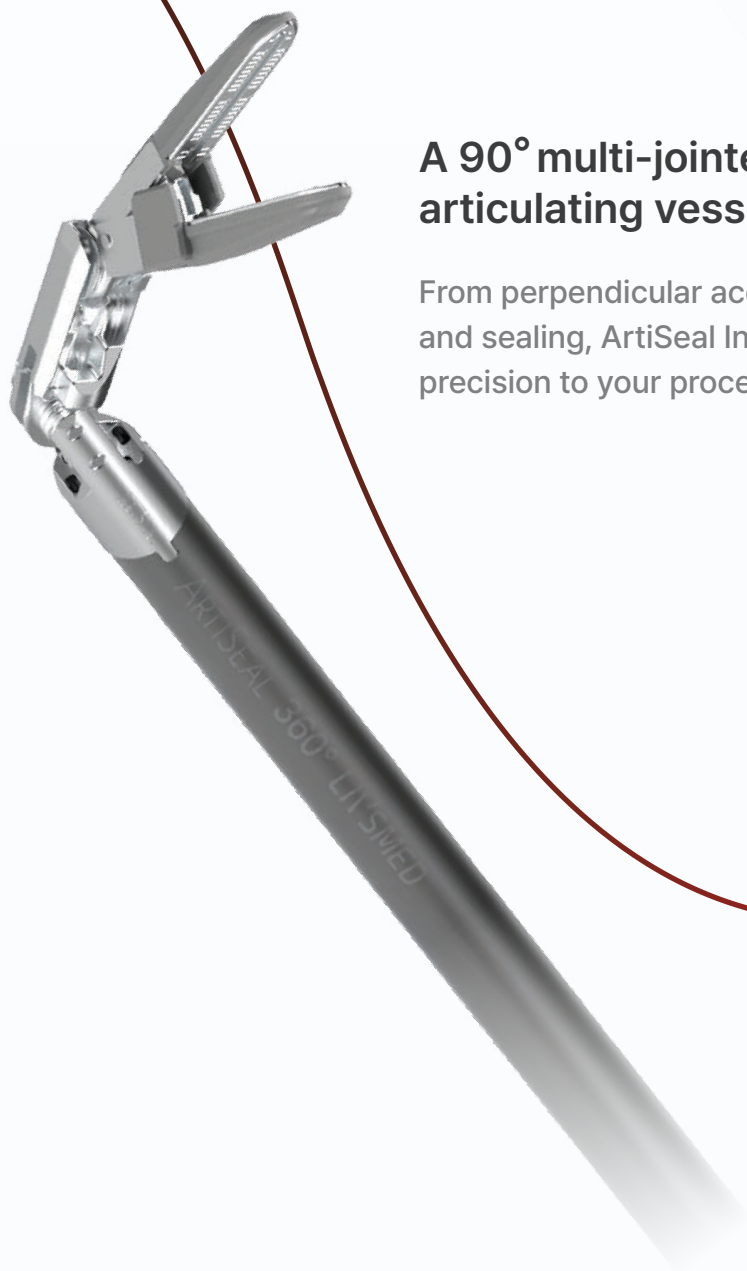

A 90° multi-jointed articulating vessel sealer

From perpendicular access to confident cutting
and sealing, ArtiSeal Instruments bring greater
precision to your procedures

LIVSMED

www.livsmed.us

2305 Historic Decatur Rd, Suite 100

San Diego, CA 92106

TEL. 619-872-5555

EMAIL. infoUSA@livsmed.com



Homepage

26103v.02